



TheraCal PT™

Kahelikõvastunud vaigu baasil kaltsiumsilikaat
pulpotoomiaks

4 g, 89€ + KM

Peamised näidustused:

1. Pulpotoomia

Sekundaarsed näidustused :

1. Otseseks pulbi katmiseks

- Kaltsiumi vabanemine ainulaadse hüdrofilise maatriksi töttu
- Süstla pakend – materjali mugav segamine ja aplitseerimine
- Kahelikõvastumine tagab täieliku tugevuse ja kiire töö
- Suurepärane röntgenkontrast
- 11 pH säilitab pulbi vitaalsuse
- Vees kergesti lahustuv, talub hästi niiskust
- Tööaeg - 45 s, kövenemisaeg - 5 min.
- Taastamise saab lõpetada ühe visiidiga, kohe pärast TheraCal PT kövenemist



Pulpotomy Technique

1




Perform pulpotomy treatment.

2



Achieve hemostasis.

3

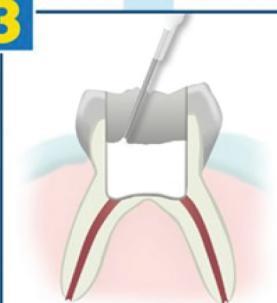

Place TheraCal PT directly in the pulp chamber, and ensure good adaptation to the cavity walls and margins.

4



Light cure TheraCal PT for 10 seconds. Place desired adhesive, base, and/or restoration following manufacturer's directions for restoring tooth.

LISAINFO TOODETE KOHTA

DAB Eesti OÜ

Tulika 19 c-4 k, Tallinn

Tel: 6 39 13 20 / 21 / 25 / 27 / 29

Tel: 8 00 33 00 (tasuta / ainult lauatelefonilt)

www.dabdental.ee